

How to Program Your Agency Name into your LifePak 15 Monitor

1. Enter Setup Menu - - [hold down “Options” and “Event” keys while powering up the LP15.
2. Press Speed Dial knob 4 times , this will open the Setup Menu if no password was programmed.
3. Choose GENERAL from the Setup screen - - [Press Speed Dial knob when “General” is highlighted]
4. Choose DEVICE ID from the General screen - - [Press Speed Dial knob when Device ID is highlighted]
5. Spell your agency by scrolling through then choosing the alpha-numeric characters on the screen – [Press END when finished]
6. Turn Off monitor - [Press ON key for two seconds]
7. Please call technical support at 800-442-1142 for assistance.