


12 Lead Placement

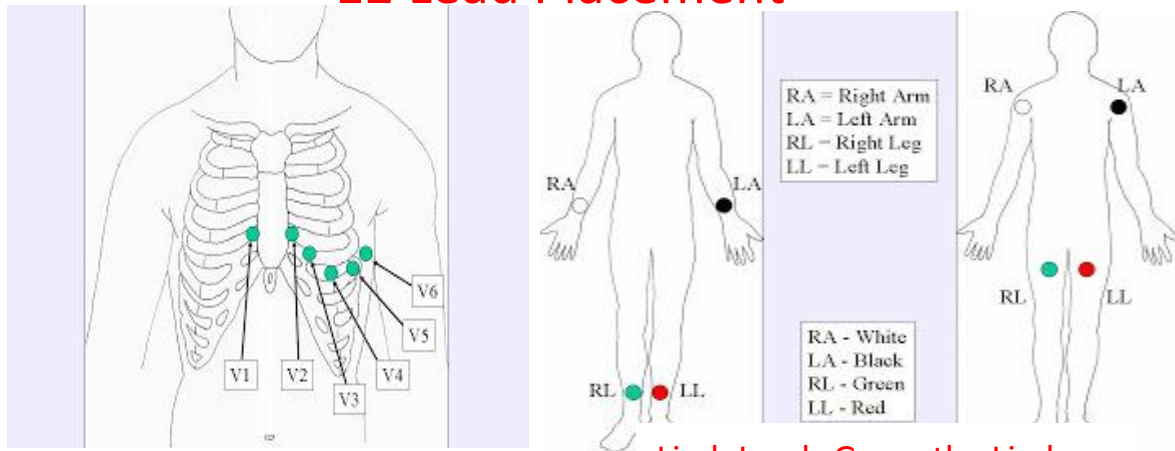

Limb Leads Go on the Limbs

ECG TRANSMISSION

- SUSPECTED ACUTE CORONARY SYNDROME (ACS)?
- ACQUIRE A 12 LEAD & TRANSMIT FOR ALL PATIENTS WITH PAIN OR DISCOMFORT BETWEEN THE CHIN AND PELVIS
- CALL RECEIVING HOSPITAL TO CONFIRM RECEIPT OF TRANSMISSION
- TRANSMIT WITHOUT DELAY
- GOAL TO HAVE PATIENT TREATED WITH LYTIC THERAPY WITHIN 30 MINUTES OF ENTERING EMERGENCY DEPARTMENT IF TIME TO PCI WILL BE >120 MINUTES FROM FIRST MEDICAL CONTACT

Phillips MRx Transmission Instructions

1. Turn the therapy knob to "Monitor"
2. Enter the Patient ID, Age, and Sex through the "Patient ID Menu" using the "Menu Select and Navigation Buttons"
3. Confirm the patient's pacing status is correct
4. Depress the "12 lead" soft key
5. Check the signal quality of each lead on the screen
6. Press the "Start Acquire" soft key – the message Acquiring ECG will appear on the screen while the monitor acquires data
 - Keep the patient still while "Acquiring ECG" is displayed
7. Following analysis, the 12 Lead Report is displayed, printed and stored internally
8. Press "Exit 12 lead" to exit the 12 lead function
9. Transmitting to a Configured Site using Bluetooth
 - Press the "Menu Select"
 - Using the "Navigation" button if necessary, select "Send" from the "12 lead Main Menu" and press "Menu Select"
 - Select the "Destination Site" from the configured list
 - Press "Menu Select" to complete the selection
 - Select the "Transmission Device" from the configured list
 - Press the "Menu Select" to complete the selection