


12 Lead Placement

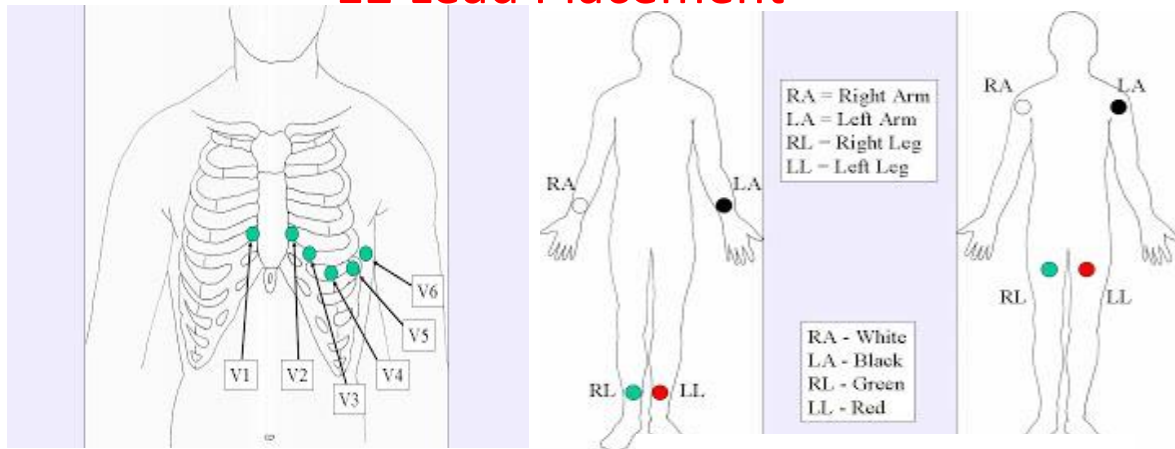

Limb Leads Go on the Limbs

ECG TRANSMISSION

- **SUSPECTED ACUTE CORONARY SYNDROME (ACS)?**
- **ACQUIRE A 12 LEAD & TRANSMIT FOR ALL PATIENTS WITH PAIN OR DISCOMFORT BETWEEN THE CHIN AND PELVIS**
- **CALL RECEIVING HOSPITAL TO CONFIRM RECEIPT OF TRANSMISSION**
- **TRANSMIT WITHOUT DELAY**
- **GOAL TO HAVE PATIENT TREATED WITH LYTIC THERAPY WITHIN 30 MINUTES OF ENTERING EMERGENCY DEPARTMENT IF TIME TO PCI WILL BE >120 MINUTES FROM FIRST MEDICAL CONTACT**

Physio-Control LifePak 15 Transmission Instructions

1. Depress the soft button on the bottom left "12 Lead"
2. Turn large dial and select "Patient Age " depress the round button to select the age
3. The monitor will display "Acquiring ECG" during the data collection process
 - a. Keep the patient still during this phase to decrease artifact
4. After the monitor has acquired and analyzed the ECG it will print a paper copy and store the data file internally
5. After the 12 Lead ECG is "Acquired" on the screen depress the soft button on the bottom left "Transmit"
6. Query the on screen Receiving Sites/Destinations by turning the large dial
 - a. NOTE: Default site has been set as nearest hospital receiving your EMS patients
 - b. "ECG 1" is the last ECG acquired by the monitor
7. Select "Site" destination by depressing the large dial
8. Select "Send" from the on screen options
9. Once the ECG has been sent and successfully received by the site selected a confirmation printout will appear